

INVACARE®  
**RELIANT™ 450**  
LIFTS



**Invacare® Reliant™ Lifts** were conceived to help reduce the possibility of caregiver back injury and to ensure dignity in patient handling. These high-quality lifts are comfortable, reliable and integral tools for caregivers. Invacare Reliant lifts support up to 450 lb., making potentially challenging transfers smooth for the patient and caregiver. The high lift range allows easy access to chairs, commodes, floors and beds (with at least 4.5" under-bed clearance). The sturdy and practical design of Reliant lifts provides the solution you need for success in the quest to provide safe and effective health care.

## Invacare® Reliant™ Lift Features

Height range allows lifting from floor and high surfaces

Low friction casters significantly improve rollability and contribute to caregiver safety

Removable battery pack eliminates downtime and simplifies the charging process

Manual lowering for instant response to power loss

Electronic system includes two 24-volt batteries: one for lift, one for battery-charging station; a manual emergency lowering device; and an anti-entrapment feature if boom meets any resistance while lowering

Available with power opening base option (model # RPL450-2)

Multi-grip push handle is angled to minimize wrist strain

Base legs adjust easily and lock securely into open position with the padded shifter handle or optional power option

Total pinch-point protection covers all moving parts for added safety

Padded swivel bar with 360° rotation and six-point hook-up allows use of Invacare chain-free slings as well as older style slings with chains or straps

Attractive neutral design blends well into any environment

Covered base provides protection against wear, dirt and moisture

Manufactured in an ISO13485 facility



### 1 Rear Locking Brakes

Locking feature on 5" rear casters is standard on all models

### 2 Manual Lowering

Manual emergency lowering gently and safely lowers patient in case of total power loss

Essential safety feature

Manual lowering is easy and effortless. Simply insert the end of a ball point pen into the hole marked "emergency" or lift on the accessible ring-pull

### 3 Invacare® Reliant™ Digital Scale

Retrofits on all Reliant 450/600 lifts  
Model # 9805 requires Reliant swivel bar

Easy to operate, four-button display

Automatic shut-off feature prolongs battery life

Two-year limited warranty

### 4 Hand Pendant

Ergonomic control with heavy-duty cord

Convenient pendant hook allows attachment and storage on multiple areas of lift

### 5 Electronics

Removable battery box conveniently located on mast features an emergency off button and beeps when battery needs recharged. Requires one 24-volt battery

Quick-release battery eliminates downtime

Anti-entrapment feature stops motion if boom meets resistance while lowering

### 6 Charging Station

Battery can be replaced immediately with second battery in wall-mounted charger

UL listed and CSA certified

Model #	Description
1075222	Reliant touch-up paint
1078276	Reliant replacement batteries
1143629	Reliant swivel bar kits
1143630	Extra-wide Reliant swivel bars
1079810	Reliant low base replacement kit (manual)
1088911	Reliant sling holder bag (four-pack)
RLS6	Reliant digital scale

---

**Invacare® Reliant™ 450 Hydraulic Lift Features**

Model # RHL450-1

Pump mechanism allows transfers of up to 450 lb. with minimal effort

Comfortable handle placement for ease of use

Wide base for maximum stability

Low maintenance

Limited warranty includes three years on lift, one year on pump

4.5" Under bed clearance

**Invacare® Reliant™ 450 Battery-Powered Lift with Low Base Features**

Model # RPL450-1

Removable battery easily charges overnight on wall-mounted charging station

Equipped with a quiet, long-life motor

Wide lift range for easier transfer

Wide base for maximum stability

4.5" Under bed clearance

Limited warranty includes three years on lift, one year on battery

150-300 lifts per charge\*

---

**HCPCS Code: E0635**

---

**Invacare® Reliant™ 450 Power Lift with Power Opening Low Base Features**

Model # RPL450-2

Removable battery easily charges overnight on wall-mounted charging station

Equipped with a quiet, long-life motor

Wide lift range for easier transfer

Wide base for maximum stability

4.5" Under bed clearance

Limited warranty includes three years on lift, one year on battery








150-300 lifts per charge\*

---






**HCPCS Code: E0635**

---

All references to HCPCS codes: Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further details.

Model #	 Height At Sling Hook Up	 Base Dimensions	 Casters	 Weight	 Weight Capacity	 Low Battery Alarm	 Emergency Controls
RHL450-1	25"-71"	4.5"H Clearance 41"W Open 26.5"W Closed 48"L	3" Front 5" Rear	93 lb.	450 lb.	N/A	N/A
RPL450-1	24"-74"	4.5"H Clearance 41"W Open 26.5"W Closed 48"L	3" Front 5" Rear	106 lb.	450 lb.	Audio	Stop and Down
RPL450-2	24"-74"	4.5"H Clearance 41"W Open 26.5"W Closed 48"L	3" Front 5" Rear	109 lb.	450 lb.	Audio	Stop and Down

	 Battery	 Charge Time	 Charger Input	 Charger Output	 Limited Warranty
RHL450-1		N/A	N/A	N/A	3 Years Lift 1 Year Pump
RPL450-1	24V DC Rechargeable 1 w/Unit	N/A	N/A	24V DC	3 Years Lift 1 Year Electronics
RPL450-2	24V DC Rechargeable 1 w/Unit	N/A	N/A	24V DC	3 Years Lift 1 Year Electronics

Complete instructions for lift usage and sling procedures are found in the Invacare Operating Manual.  
Only Invacare slings and accessories should be used on Invacare lifts.

INVACARE®  
**RELIANT™ 600**  
HEAVY-DUTY LIFT



Designed to meet the special needs of your patients weighing up to 600 lb., the **Invacare® Reliant™ 600 Heavy-Duty Lift** is secure and sturdy, providing a safe alternative to manual lifting. Both durable and functional, the Reliant 600 Lift is also a reassuring and comfortable patient lift. When answering the needs of every patient is your priority, this is your wisest choice.

---

### Invacare® Reliant™ 600 Lift Features

High lift range allows transfer to high surfaces  
Low friction casters significantly improve rollability and contribute to caregiver safety  
Designed to meet the needs of the growing bariatric population up to 600 lb.  
Manual lowering for instant response to power loss  
Unique sling fabric designed to provide the utmost in comfort and strength

Available with power opening base (model # RPL600-2)  
Essential device for safe patient handling in hospitals and LTC (long-term care) facilities  
Limited warranty includes three years on lift, one year on electric components  
Rear locking casters are standard  
Manufactured in an ISO13485 facility

---

**HCPCS Code: E0635**




RPL600-1: Powered with low base



RPL600-2: Powered with power opening low base

---

All references to HCPCS codes: Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further detail.

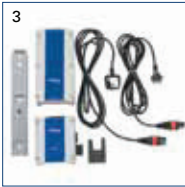

### 1. **Manual Lowering**

Manual emergency lowering gently and safely lowers the patient in case of total power loss

Essential safety feature

Manual lowering is easy and effortless. Simply insert the end of a ball point pen into the hole marked "emergency" or lift on the accessible ring-pull

### 2. **Invacare® Reliant™ Digital Scale**

Retrofits on all Reliant 450/600 lifts. Model # 9805 requires Reliant swivel bar

Easy to operate, four-button display

Automatic shut-off feature prolongs battery life

Two-year limited warranty

### 3. **Electronics**

Removable battery box conveniently located on mast features an emergency off button and beeps when battery needs recharged. Requires one 24-volt battery


Quick-release battery eliminates downtime

Anti-entrapment feature stops motion if boom meets resistance while lowering

### 4. **Charging Station**

Battery can be replaced immediately with second battery in wall-mounted charger



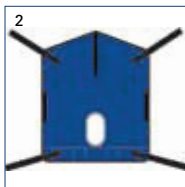




UL listed and CSA certified








### 1. **Mesh sling model # R140**

### 2. **Mesh sling with commode opening model # R141**




Model #	 Height At Sling Hook Up	 Base Dimensions	 Casters	 Weight	 Weight Capacity	 Battery Life Indicator	 Emergency Controls
RPL600-1	28"-68" H	4.5"H Clearance 41"W Open 26.5"W Closed 48"L	3.5" Front 5" Rear	112 lb.	600 lb.	N/A	Stop and Down
RPL600-2	28"-68" H	4.5"H Clearance 41"W Open 26.5"W Closed 48"L	3.5" Front 5" Rear	115 lb.	600 lb.	N/A	Stop and Down

	 Battery	 Charge Time	 Charger Input	 Charger Output	 Limited Warranty
RPL600-1	N/A	N/A	N/A	N/A	3 Years Lift 1 Years Electronics
RPL600-2	N/A	N/A	N/A	N/A	3 Years Lift 1 Years Electronics



INVACARE®  
**9805P**  
HYDRAULIC LIFT

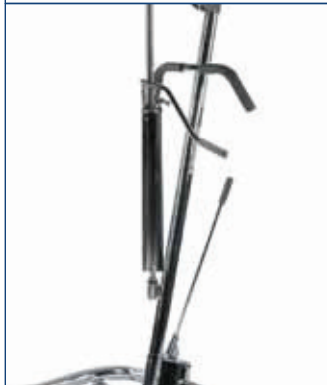

Too often, manual patient transfers lead to caregiver back injuries. The **Invacare® 9805P Hydraulic Lift** was created to help make handling situations safer and affordable for everyone involved. With their slim design, these hydraulic lifts are versatile enough to use for most any and all patient transfers. The features of the 9805P include a padded swivel bar and push handle, a 450 lb. weight capacity, durable chrome-plated construction or painted finish and 360° patient rotation without side-to-side sway. Lightweight construction and easy disassembly allow for quick transport and setup. Indispensable to patients and caregivers alike, the 9805P lifts offer safety, comfort and stability.

**Features**

Six-point swivel bar attachment easily adapts to all sling styles and positions

Low friction casters significantly improve rollability and contribute to caregiver safety

20"-64" range allows for patient to be picked up from lying position on floor

Pump handle can rotate from side to side for convenience of attendant

Offset mast and boom style provide better lift path, maximizing full travel range

Swivel bar and pump handle are padded for comfort

Elongated handle design serves a broad range of operator heights and offers improved handling ability

Completely enclosed cam moves legs of adjustable base

Base offers obstruction-free operation; ideal for cleaning

Screw-in pin at mast base provides maximum stability and prevents removal of mast

**HCPCS Code: E0630**



- 1. Adjustable base Model # 9884
- 2. Hydraulic pump
- 3. Swivel bar
- 4. Standard sling

Model #	Chain or Strap Assembly Kit (required for Invacare 9000 Series Wheelchairs)
9070	sling strap assembly kit
9071	sling chain assembly kit

Model #	Fabric/Style
9042	solid polyester without commode
9043	solid polyester/with commode
9046	mesh polyester/without commode
9047	mesh polyester/with commode



Model #	Height At Sling Hook Up	Base Dimensions	Casters	Weight	Weight Capacity	Limited Warranty
9805P	20"-64" H	6.5"H Clearance 42.25"W Open 22"W Closed 46.75"L	5"	68 lb.	450 lb.	3 Years Lift 1 Year Hydraulic Pump

INVACARE®  
**RELIANT™ 350**  
STAND-UP LIFT



**The Invacare® Reliant™ Series Stand-Up Lifts** are an ideal product for everyday patient handling, ensuring caregiver security and patient peace of mind during the transfer. Intended for use with weight-bearing patients as well as those needing rehabilitation support, the Invacare Reliant Stand-Up lift will be a valuable addition to your safety management system.

## Features

Innovative adjustment features adapt to a wide variety of body sizes and shapes

Low friction casters significantly improve rollability and contribute to caregiver safety

Multi-function slings allow for quick toileting, stand assist or full-support seated transfer

Adjustable leg support features a soft pad with lateral contour that secures legs in three positions for comfort at any height

Wide, non-slip footplate provides stable base for transfer

Under-bed clearance of 4.5" allows accessibility to most nursing home, hospital and homecare bed styles

Available with power opening base (Model # RPS350-2)

Removable battery eliminates downtime



Ergonomic features make it ideal for toilet transfers, as well as for dependent and rehabilitation purposes

Features include fast, simple sling attachment that protects against accidental disengagement; wide push handle; good access for clothing removal; and comfortable slings with extended back support that prevent pressure in the under-arm area

Adjustable leg strap option secures leg against knee pad for optimum positioning on footplate

Manufactured in an ISO13485 facility

## HCPCS Code: E0635

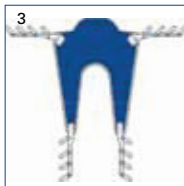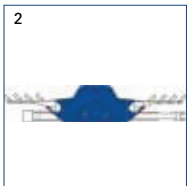

1. Electronics
2. Extra-large standing sling\* Model # R130
3. Extra-large transfer sling\* Model # R134

\*Complete instructions for lift usage and sling procedures are found in the Invacare operating manual. Only Invacare® slings and accessories should be used on Invacare® lifts.



Model #	Height At Sling Hook Up	Base Dimensions	Casters	Weight	Weight Capacity	Low Battery Alarm	Emergency Controls
RPS350-1	39.6"-63.7" H	4.5"H Clearance 36.8"W Open 25.7"W Closed 35.5"L	3" Front 5" Rear	115 lb.	350 lb.	Audio	Stop and Down
RPS350-2	39.6"-63.7" H	4.5"H Clearance 35.8"W Open 25.7"W Closed 35.5"L	3" Front 5" Rear	118 lb.	350 lb.	Audio	Stop and Down



	Battery	Charge Time	Charger Input	Charger Output	Limited Warranty
RPS350-1	N/A	N/A	N/A	N/A	3 Years Lift 1 Year Motor
RPS350-2	N/A	N/A	N/A	N/A	3 Years Lift 1 Year Motor

All references to HCPCS codes: Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further detail.





# INVACARE® SLINGS



Invacare offers a complete line of slings to meet the dynamic needs in both the long-term care and homecare settings. **Invacare® Reliant™ Series Slings** are an industry standard, and deliver the ease of use, comfort and security that is standard in all Invacare Products. The **Invacare® Premier Series Slings** offer additional fabric and style options to meet a wider array of needs, as well as value-adding features for the caregiver and patient alike.

**Invacare® Reliant™ Slings** come in an array of styles to meet a variety of client needs. Reliant slings feature a unique, fully padded fabric that forms a soft yet supportive surface. The soft, brushed lining helps prevent sliding for ultimate comfort and skin protection, and the stretch-resistant feature helps the slings hold their shape for superior support and ease of positioning. For clients and caregivers alike, Invacare Reliant slings are designed with comfort and safety in mind.

**Features**

Sling size and fit can vary significantly depending on patient weight and girth. These are general guidelines. Consult physician before sling selection.





Washing instructions are on each sling label. Washing temperatures should not exceed 180° F. Air-dry or dry at low temperatures.

All slings are constructed of polyester fabrics which can be used in wet or dry environments.

All Invacare® slings were designed specifically for use with Invacare® brand lifts for proper usage and optimal performance. To receive the maximum benefits from Invacare products, please see the size guide, product specs and operator manuals for solutions that best suit your needs.

**Sling Sizes**


Color/Weight Chart

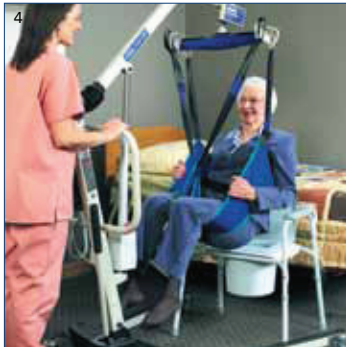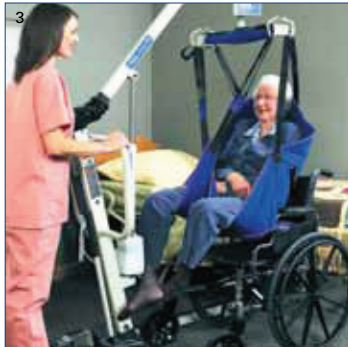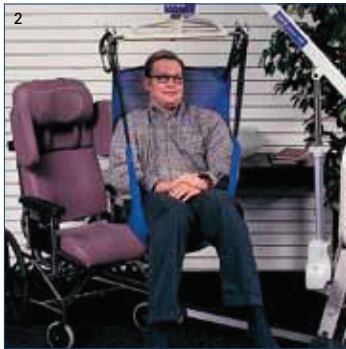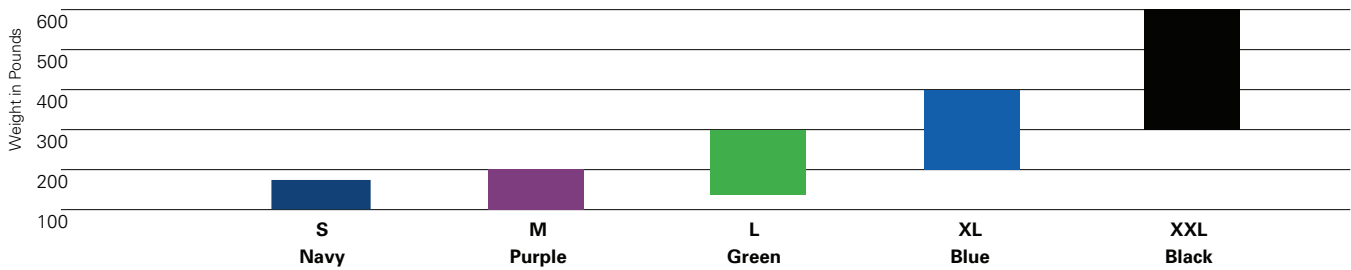

- 1 **Invacare® Reliant™ Full-Body Slings**
- 2 **Invacare® Reliant™ Full-Body Slings with Commode Opening**
- 3 **Invacare® Reliant™ Full-Body Slings with Commode Opening**
- 4 **Invacare® Reliant™ Toileting Sling with Belt**

Complete instructions for lift usage and sling procedures are found in the Invacare Operating Manual. Only Invacare slings and accessories should be used on Invacare lifts.



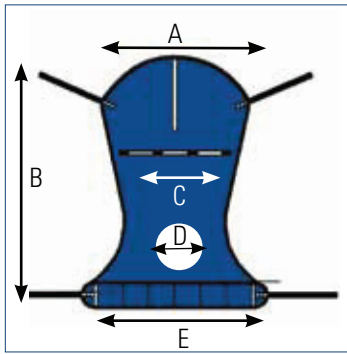


### Full Body Sling With/Without Commode Opening (CO)

An easy to use while transferring the patient from the bed to a wheelchair, toilet or shower chair or from floor to bed. It offers total head and neck support and the padded lower section provides additional under-thigh comfort and support. Use the full body sling for patients who are totally dependent, partially-dependent, non-weight bearing, very heavy or have limited head control. Use this sling for amputee patients (in a reclined position).

Compatible with the following lifts

Invacare® Reliant™ 450, 600 Lifts

SIZE	A	B	C	D	E	Weight Capacity	Model #/Material
M	41.4"	53.3	23.9"	N/A	43.7"	450 lb.	R110/Mesh
L	45.5"	58.3"	26.3"	N/A	47.7"	450 lb.	R111/Mesh
XXL	45"	55"	45"	N/A	45"	600 lb.	R140/Mesh
M	41.5"	53.3"	23.9"	6"	43.7"	450 lb.	R114/Mesh w/CO
L	45.5"	58.3"	26.3"	7"	47.7"	450 lb.	R115/Mesh w/CO
XL	45.5"	63.2"	26.3"	7"	47.7"	450 lb.	R116/Mesh w/CO
XXL	45"	55"	45"	7"	45"	600 lb.	R141/ Mesh w/CO
M	41.5"	54.7"	23.2"	N/A	41.8"	450 lb.	R112/Solid Fabric
L	45.5"	60.3"	26.3"	N/A	45"	450 lb.	R113/Solid Fabric
XL	45.5"	65.3"	26.1"	N/A	45"	450 lb.	R117/Solid Fabric

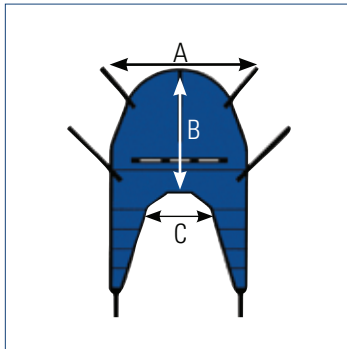


### Divided Leg Sling

The divided leg sling is a secure, easy-to fit general purpose sling, designed for the majority of sling users. Use the divided leg sling for patients who are totally dependent, partially dependent, non-weight bearing, very heavy or have limited head control. Use this sling when transferring the patient from the bed to a wheelchair, recliner or shower chair. The large opening also accommodates toileting.

Compatible with the following lifts

Invacare® Reliant™ 450, 600 Lifts

SIZE	A	B	C	Weight Capacity	Model #
S	33"	29.16"	12"	450 lb.	R100P
M	37.5"	35.66"	17"	450 lb.	R100
L	41"	40.66"	20.5"	450 lb.	R101
XL	44.5"	43.16"	22.6"	450 lb.	R102

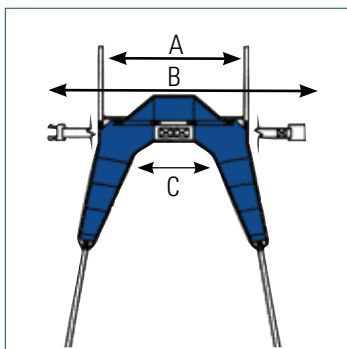


### Solid Fabric Toileting Sling

The Toileting Sling offers superb access for quick toileting and hygiene. It is easy to place and remove in a bed or chair situation. The fully padded design supports the back, offers under thigh comfort and allows for removal of clothing. Color coded positioning straps help align the sling properly on the hanger bar. Note: This sling requires the patient to have FULL head and neck control.

Compatible with the following lifts

Invacare® Reliant™ 450, 600 Lifts

SIZE	A	B	C	Weight Capacity	Model #
L	36"	44.7"	20.5"	450 lb.	R121

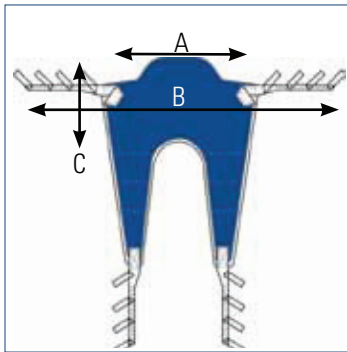

### Transport Sling (use with Stand-Up Lift)


Transport slings are for transferring patients with minimal weight-bearing capacity, who have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance) and are able to bend at the hips, knees and ankles. Use the R134 and R136 transport sling for transferring patient from bed to chair, chair to car or general in-room transport. Do not raise patient to a full standing position while using the transport sling.

Compatible with the following lifts

Invacare® Reliant™ 350 Lift

Invacare® Get-U-Up™ Lift

SIZE	A	B	C	Weight Capacity	Model #
One Size	34"	13.5"	9"	440 lb.	R134
XL	46"	13.5"	10"	440 lb.	R136

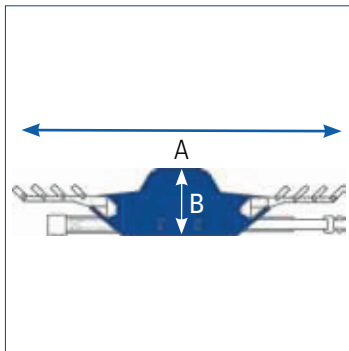

### Standing Sling (use with Stand-Up Lift)

Standing slings are for patients who are partially dependent, have at least 60% weight bearing capacity, have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance) and are able to bend at the hip, knees and ankles. Use the R130 or R131 standing slings for standing assistance, quick toileting, weight bearing practice and for transferring patient from bed to chair, or chair to bed.

Compatible with the following lifts


Invacare® Reliant™ 350 Lift

Invacare® Get-U-Up™ Lift

SIZE	A	B	Weight Capacity	Model #
One Size	60"	16"	440 lb.	R130
XL	84"	16"	440 lb.	R131


## Warning!

BEFORE using the Patient Lift, READ and UNDERSTAND the Operator's Manual for proper operation and safety procedures.

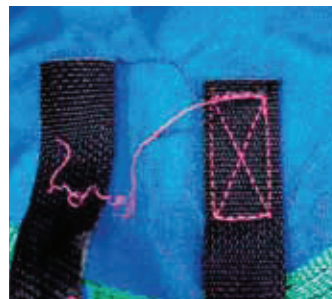

Use the sling that is recommended by the individual's doctor, nurse or medical attendant. Before lifting, check all sling straps for secure points of attachment on lift device. Do not exceed weight limitation posted on lift. Use only with Invacare Patient Lifts. Bleached, torn, cut, frayed or broken slings are unsafe and could result in injury. Discard immediately. Do not alter slings. Use only on Invacare lifts.

The **Invacare® Premier Series Slings** balance solutions for any transfer. Invacare says this with confidence because these slings were developed in collaboration with Occupational and Physical therapists. Invacare Premier slings are available in a wide variety of styles and can be adjusted to ensure a comfortable and secure fit for the patient. Clearly labeled slings and easy attachment points make it simple for the caregiver to quickly attach the sling and lift the patient.

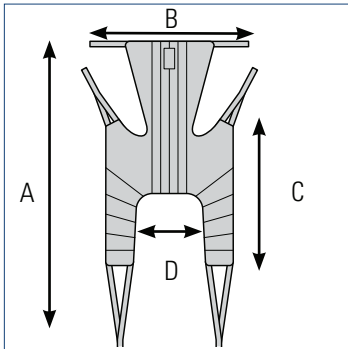


**Universal High SlingPlus**

Supports the whole body and head in transfers to/from seated or lying position. Offers a comfortable recline for those with limited head, trunk or hip control. Allows arms to hang outside the sling.

Compatible with the following lifts  
 Invacare® Jasmine™ Lift  
 Invacare® Reliant™ 450, 600 Lift

	Model #/Material				
SIZE	A	B	C	D	Polyester Fabric
S	54.4"	33.1"	34.7"	13.4"	2485960
M	59.5"	37.4"	36.6"	17.7"	2485961
L	62.3"	43.3"	38.2"	22.1"	2485962
XL	62.6"	46.9"	38.2"	24.8"	2485963

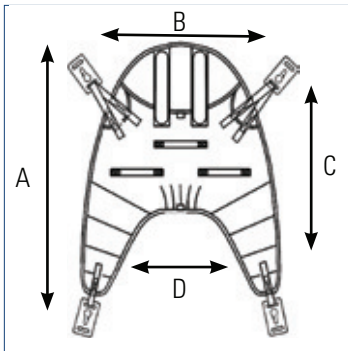


**Cradle Sling**

Designed for use with the Jasmine lift. Comfort and support of the whole body and head are crucial for those with head, trunk and hip concerns. 4-point attachment clips onto the cradle bar for best positioning in horizontal-to-seated or seated-to-seated transfers. Head support is optional by removing rigidizers.

Compatible with the following lift  
 Invacare® Jasmine™ Lift with cradle accessory

	Model #/Material				
SIZE	A	B	C	D	Polyester Fabric
S	42.6"	33.1"	28.8"	15.0"	3483777
M	55.9"	39.4"	32.7"	15.8"	3483778
L	61.1"	43.3"	36.2"	20.1"	3483779

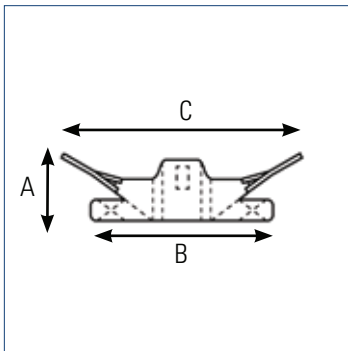


**Stand Assist Sling**

Designed for use with the Roze and Reliant 350 and 440 lifts, the Stand Assist Sling offers a comfortable and safe transfer for those who have some weight bearing ability, but cannot stand independently. The sling is well-padded and features non-slip material on the side that touches the patient to assure a comfortable transfer position.

Compatible with the following lift  
 Invacare® Roze™ Lift  
 Invacare® Reliant™ 350 Lift

	Model #/Material			
SIZE	A	B	C	Polyester Fabric
S	10.2"	33.1"	34.7"	2484685
M	13.0"	35.3"	46.1"	2484686
L	14.2"	37.4"	50.0"	2484687
XL	14.2"	40.2"	71.3"	2484688

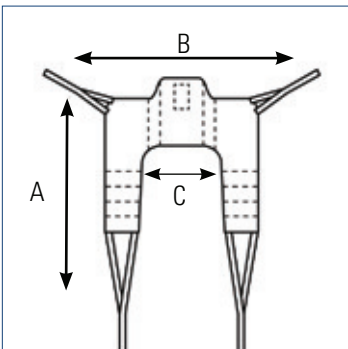


**Transfer Stand Assist Sling**

Also designed for optimal use with the Roze and Reliant 350 and 440 lifts, this high-quality sling provides simple seated transfer for patients with minimal weight bearing ability.

Compatible with the following lifts  
 Invacare® Roze™ Lift  
 Invacare® Reliant™ 350 Lift

	Model #/Material			
SIZE	A	B	C	Polyester Fabric
S	33.9"	33.1"	9.5"	2484689
M	40.2"	35.3"	10.2"	2484690
L	46.5"	37.4"	13.0"	2484691
XL	48.5"	40.2"	13.8"	2485117

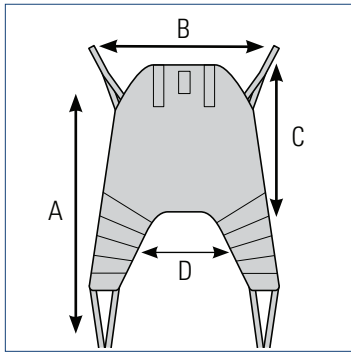


### Universal High Sling

Supports the whole body and head in transfers to/from seated or lying position. Offers a comfortable recline for those with limited head, trunk or hip control. Universal High Sling allows arms to hang outside the sling.

Compatible with the following lifts  
 Invacare® Jasmine™ Lift  
 Invacare® Reliant™ 450, 600 Lift

SIZE	A	B	C	D	Model #/Material	
					Polyester Fabric	Spacer Fabric
S	42.6"	33.1"	28.8"	15.0"	2483777	2485956
M	55.9"	39.4"	32.7"	15.8"	2483778	2485957
L	61.1"	43.3"	36.2"	20.1"	2483779	2485958
XL	66.2"	45.3"	36.2"	21.7"	2485949	2485959

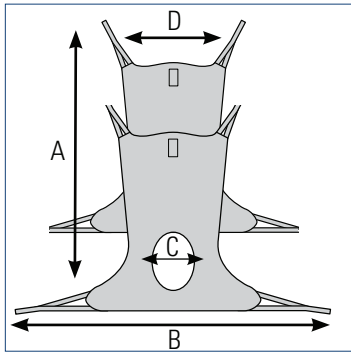


### Comfort Sling and Comfort Sling with Commode Opening

Amputees and residents with chronic or postural pain have delicate needs that may require a constant seated position. Comfort Slings transfer them from a horizontal to seated position, with an additional model if toileting is also required.

Compatible with the following lifts  
 Invacare® Jasmine™ Lift  
 Invacare® Reliant™ 450, 600 Lift

SIZE	A	B	C	D	Model #/Material		
					Comfort Sling Net Fabric	Comfort Sling w/ Toileting Opening Polyester Fabric	Comfort Sling w/ Toileting Opening Net Fabric
S	45.3"	41.4"	21.7"	40.2"	2485968	2451092	2451097
M	55.2"	46.1"	24.0"	46.5"	2485969	2451093	2451098
L	57.1"	50.8"	26.4"	50.4"	2485970	2451094	2451099
XL	57.5"	52.4"	29.6"	57.9"	2485947	2451095	2451100

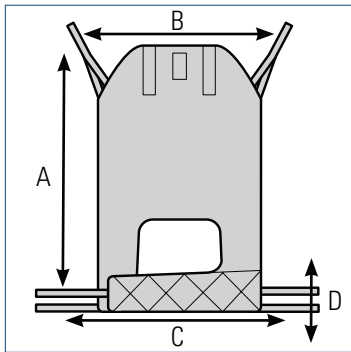


### Easy-Fit Sling

For those with more challenging physical disabilities, including amputees, this sling supports the whole body and head in an upright position with separate leg supports and optimal hip position.

Compatible with the following lifts  
 Invacare® Jasmine™ Lift  
 Invacare® Reliant™ 450, 600 Lift

SIZE	A	B	C	D	Model #/Material	
					Polyester Fabric	
S	44.5"	29.9"	25.2"	7.1"	2451102	
M	47.3"	31.5"	26.6"	7.5"	2451103	
L	50.0"	33.9"	28.4"	8.3"	2451104	
XL	54.8"	37.0"	30.7"	8.7"	2451105	

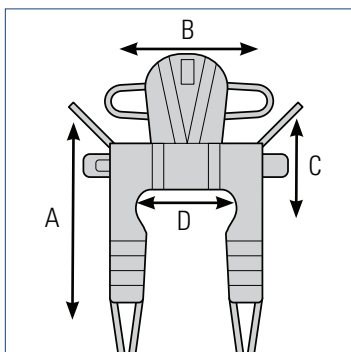

### Dress Toileting High Sling

Featured belt offers secure support and positioning. Easy-to-apply Dress Toileting High Sling is suitable for seat-to-seat transfers and dressing or undressing during transfers to toilet. Dress Toileting High Sling is specially designed for users with reduced head control.

Compatible with the following lifts  
 Invacare® Jasmine™ Lift  
 Invacare® Reliant™ 450, 600 Lift

SIZE	A	B	C	D	Model #/Material	
					Polyester Fabric	
S	34.3"	28.0"	7.9"	16.9"	2478444	
M	37.4"	31.5"	9.1"	19.3"	2478445	
L	40.2"	35.5"	9.1"	23.6"	2478446	
XL	43.3"	42.6"	10.6"	29.9"	2484780	

